

Diagnostic Kit for Gastrin 17

(Fluorescence Immunochromatographic Assay)

Instructions for Use

INTENDED USE

This kit is intended for in vitro quantitative detection the content of Gastrin 17 (G-17) in human serum/plasma/whole blood samples. This kit only provides the test result of Gastrin 17 (G-17). The obtained result shall be analyzed in combination with other clinical information. It must only be used by healthcare professionals.

SUMMARY

Gastrin, also known as pepsin, is a gastrointestinal hormone mainly secreted by G cells of gastric antrum and duodenum and plays an important role in regulating digestive tract function and maintaining intact structure of digestive tract. Gastrin can promote gastric acid secretion, facilitate growth of gastrointestinal mucosal cells, and improve nutrition and blood supply of mucosa. In human body, more than 95% of biologically active gastrin is α -amidated gastrin, which mainly contains two isomers: G-17 and G-34. G-17 shows the highest content in human body (about 80%~90%). Secretion of G-17 is strictly controlled by pH value of gastric antrum and shows negative feedback mechanism that relative to gastric acid.

PRINCIPLE OF DETECTION

This kit uses the double-antibody sandwich reaction principle with high specificity and fluorescence immunochromatography to quantitatively detect amount of the G-17 antigen in the human serum / plasma / whole blood samples. The reagent strip contains anti-G-17 antibody pre-fixed on the test line (T) of the membrane and goat anti-rabbit IgG antibody in the control line (C). The label pad contains pre-coated fluorescently labeled anti-G-17 antibody and rabbit IgG antibody. During sample testing, the G-17 antigen in the sample binds with fluorescence-labelled anti-G-17 antibody to form immune complex. Under the immunochromatographic effect, the complex and sample flow towards the absorbent paper inside the nitrocellulose membrane. When passing through the test line (T), the complex binds with the coating anti-G-17 antibody to form “anti-G-17 antibody-G-17 antigen- fluorescence-labelled anti G-17 antibody” complex and thus agglutinate. When passing through the quality control line (C), the fluorescently labeled rabbit IgG antibody binds to the coated goat anti-rabbit IgG antibody to form a “Goat anti-rabbit IgG antibody-fluorescently - Rabbit IgG antibody” complex and agglutinates. The G-17 concentration in the sample is positively correlated with the fluorescence intensity, and the concentration of G-17 in the sample can be detected by the fluorescence immune analyzer.

MAIN KIT COMPONENTS

Catalogue number	53321101	53321105	53321120	53321125
------------------	----------	----------	----------	----------

Specification Components	1 Test/Kit	5 Tests/Kit	20 Tests/Kit	25 Tests/Kit
Test Device(s)	1	5	20	25
Sample Diluents	1	5	20	25
Instructions for Use	1	1	1	1

MAIN ACTIVE INGREDIENTS

- 1 Test line (T line): T line area of nitrocellulose membrane is coated with anti-G-17 antibody.
- 2 Control line (C line): C line area of nitrocellulose membrane is coated with Goat anti-rabbit IgG antibody.
- 3 Labeling pad: It is coated with fluorescent-microsphere-labeled anti- G-17 antibody and Rabbit IgG antibody
4. Main component of sample diluent is 20mM, pH7.4 PBS solution.



Warning: The diluent includes 0.1% Proclin300

H317: May cause an allergic skin reaction.

H412: Harmful to aquatic life with long lasting effects.

P280: Wear protective gloves/protective clothing/face protection.

P333+P313: If skin irritation or a rash occurs: Get medical advice/attention.

P362+P364: Take off contaminated clothing and wash it before reuse.

STORAGE CONDITION

1. The kit should be stored at 2°C~30°C. The shelf life of the kit is 24 months.
2. Do not use the kit after the expiration date

APPLICABLE INSTRUMENT

The product can only be used with portable immune analyzer WIZ-A101 from WIZ BIOTECH.

SAMPLE COLLECTION AND STORAGE

1. This kit is indicated for testing of venous whole blood, serum, and plasma. For whole blood and plasma samples, can use anticoagulant such as EDTA-K2, heparin, and sodium citrate.
2. Any sample taken from human may be infectious and shall be disposed using standard bio-safety procedure.
3. To avoid interference with the test result, do not use hyperlipidemic, hemolytic or turbid sample.
4. Blood sample shall be collected after overnight fasting (about 10 hours).
5. Whole blood collection: According to standard blood sampling procedure, use blood collection tube containing suitable anticoagulant to collect whole blood sample by venous puncture. Whole blood shall be tested as soon as possible after collected. If test cannot be performed in time, the sample shall be stored at 2°C~8°C for up to 4 hours.
6. Serum/plasma collection: According to standard blood sampling procedure, use blood collection tube containing suitable anticoagulant to collect whole blood sample by venous puncture. Serum and plasma shall be separated as soon as possible after blood sampling to avoid hemolysis. The separated serum and plasma shall be tested immediately. If test cannot be performed in time, the sample can be stored at 2°C~8°C for 4 hours.
7. Avoid repeated freezing-thawing of sample. Turbid sample or sample with sediment shall be tested after centrifugation or filtered to clarity.
8. Before test, sample should be in room temperature and mixed thoroughly.

REAGENT PREPARATION

1. Use immediately after open the aluminum foil bag.
2. Before test, restore the reagent to room temperature.

TEST METHOD

Read the instruction for use and test operation manual completely before the test and restore the reagent to room temperature before the test. Do not perform the test without restoring the reagent to room temperature to avoid affecting the accuracy of the test results.

The following operation steps are all performed provided that WIZ-A101 portable immune analyzer has been ready for use. To select or change any setting of WIZ-A101 portable immune analyzer, please refer to user manual of WIZ-A101 portable immune analyzer.

WIZ-A101 portable immune analyzer has two test modes: I standard test mode; II quick test mode. It is required to select corresponding test mode before test, and then go into the next operation:

I Select standard test mode of WIZ-A101 portable immune analyzer

I-1: Use of portable immune analyzer

- (1) Open the aluminum foil bag package of reagent and take out the test device.
- (2) Horizontally insert the test device into the slot of immune analyzer.
- (3) On home page of operation interface of immune analyzer, click “Standard” to enter test interface.
- (4) Click “QC Scan” to scan the QR code on inner side of the kit; input kit related parameters into instrument and select sample type.

Note: Each batch number of the kit shall be scanned for once. If the batch number has been scanned, then skip this step.

- (5) Check the consistency of “Product Name”, “Batch Number” and etc. on test interface with information on the kit label.
- (6) After information consistency is confirmed, take out sample diluents, add 80μL of serum/plasma/whole blood sample, and sufficiently mix.
- (7) Add 80μL of above mixed solution into the sample hole of test device.
- (8) After complete sample addition, click “Timing” and remaining test time will be automatically displayed on the interface.
- (9) Immune analyzer will automatically complete test and analysis when test time is reached.

I-2: Result calculation and display

After the test by immune analyzer is completed, test result will be displayed on test interface or can be viewed through “History” on home page of operation interface.

II Select quick test mode of WIZ-A101 portable immune analyzer

II-1: Drip the sample into the test device

- (1) Open the aluminum foil bag package, take out the test device, and horizontally place it on examination table;
- (2) Take out sample diluents, add 80μL of serum/plasma/whole blood sample and mix;
- (3) Add 80μL of above mixed solution into the sample hole of test device;
- (4) Start stopwatch and wait for 15 minutes.

II-2: Use of portable immune analyzer

- (1) After post-sample-addition reaction for 15 minutes, horizontally insert the test device into the slot of immune

analyzer;

(2) On home page of operation interface of immune analyzer, click "Quick Test" to enter test interface;

(3) Click "QC Scan" to scan the QR code on inner side of the kit; input kit related parameters into instrument, and select sample type;

Note: Each batch number of the kit shall be scanned for once. If the batch number has been scanned, then skip this step.

(4) Check the consistency of "Product Name", "Batch Number" and etc. on test interface with information on the kit label;

(5) After information consistency is confirmed, click "Test". The immune analyzer will automatically complete test and analysis.

II-3: Result calculation and display

After the test by immune analyzer is completed, results will be displayed on test interface or can be viewed through "History" on home page of operation interface.

REFERENCE INTERVAL

1. Study of G-17 reference interval is conducted through referring to C28-A2 document published by US Clinical and Laboratory Standards Institute (CLSI)- How to Define and Determine Reference Intervals in the Clinical Laboratory - Second Edition and WST 402-2012 Define and Determine the Reference Intervals of Test Items in Clinical Laboratory. The obtained reference interval of G-17 is: 1pmol/L-7pmol/L.

2. Due to difference in geography, race, age etc., each laboratory is suggested to establish reference interval of G-17 that is suitable for local populations and has clinical significance.

INTERPRETATION OF THE RESULT

1. If the G-17 measured concentration of sample is higher than the reference value range, the physiological change or stress response and other states shall be excluded. Tangible abnormal should be diagnosed in combination with clinical symptoms.

2. Test result of this method is only applicable to evaluation using the reference value established in this method and cannot be directly compared with result of other method.

3. Other factors that may cause wrong test result include technical reason, operation error and other sample factors.

*Invalid result: If the assay result is invalid, the WIZ-A101 portable immunoassay analyzer will display an "invalid" result. The test personnel should read the kit instructions and portable immunoassay analyzer instructions carefully and repeat the test. If the "invalid" result is obtained If it reappears, please contact Xiamen Wiz Biotech Co., Ltd.

PRECAUTIONS

1. This kit can only be used for in vitro diagnosis.

2. This kit is for healthcare professionals only.

3. Before test, restore reagent and sample to room temperature.

4. This kit is disposable.

5. Do not use expired reagent.

6. Sample collection and storage must be performed in strict accordance with this instruction.

7. The reagent should be stored in strict accordance with the conditions specified in this instruction for use. Do not store the reagent under freezing condition.

8. Do not open the aluminum foil bags before test and protect the products from moisture; do not use if the aluminum foil bags are damaged or if the test reagents are wet.
9. For all components of the kit, it is recommended not to mix or interchange different batches.
10. Excessive or insufficient sample may lead to deviation of result.
11. Do not confuse sample hole with result observation window. Adding sample to result observation window will make test result invalid.
12. Test method should strictly follow the instruction for use.
13. For specific explanation of test result, analysis shall be performed in combination with clinical information.
14. The used reagent and sample shall be properly disposed as medical waste with risk of biological infection and handled safely.
15. Desiccant in aluminum foil bag is inedible.
16. Sample diluent is only used for test. Do not drink it. Wrong use may lead to biological hazard.
17. During test, test procedure, precautions and result explanation of the reagent must be followed to avoid wrong result.

PRODUCT PERFORMANCE INDEX

Evaluation performed using internal enterprise reference, that all performance indexes of this kit conform to standard. Specific performance indexes are as below:

1. Reportable range of the reagent (kit): Within 1~160 pmol/L
2. Linear range: Linearity of the reagent (kit) is within 1pmol/L~40 pmol/L;
 - a) Relative deviation of the determination result shall be within $\pm 15\%$;
 - b) The correlation coefficient (r) of linearity should be ≥ 0.9900 .
3. Accuracy: The recovery rate should be within the range of [85% ~ 115%];
4. Limit of detection: G-17 ≤ 0.5 pmol/L;
5. Repeatability: CV $\leq 15\%$;
6. HOOK effect: In case of G-17 concentration ≤ 400 pmol/L, there is no HOOK effect.
7. Specificity: When PGI and PGII are tested under the following concentration condition, there is no cross phenomenon.


Material	Concentration	Material	Concentration
PGI	200ng/mL	PGII	50ng/mL

8. Interfering substance: Following substances are found to cause no interference when they are tested at the given concentration.

Interfering substance	Concentration	Interfering substance	Concentration
Bilirubin	2.0 mg/mL	Hemoglobin	10.0 mg/mL
Triglyceride	40.0 mg/mL	Rheumatoid factor	1500.0 IU/mL

9. Clinical performance

Clinical evaluation performance of the product is assessed through collecting 155 clinical samples. The results are compared by using the corresponding kit of marketed euzymelinked immunosorbent method as the reference reagent. Use linear regression to investigate their comparability. The correlation coefficients of the two tests are $Y=0.952X+0.269$ and $R=0.9798$, respectively.

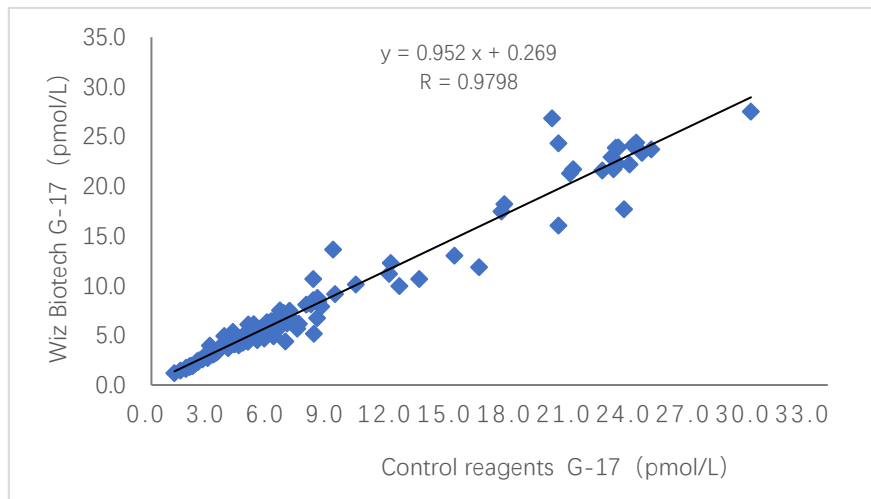










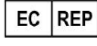

LIMITATION

1. This reagent is only used for testing human whole blood, serum, and plasma.
2. Do not agitate the sample. Insert a pipette just below the surface of the sample to collect the specimen.
3. Test shall be carried out at normal room temperature (18°C~25°C).
4. Linearity range of the kit is 1~40pmol/L. To measure accurate concentration of high-concentration sample that exceeds measurement linearity range of the kit, measurement and calculation should be performed after dilution to within the linearity range of the kit. Under the conditions of normal saline or sample diluents, the maximum dilution multiple of this kit is 4 times and the reportable range is 1~160pmol/L.
5. Due to low concentration of the analyte, this method cannot detect analyte, this will lead to result deviation.
6. Since some non-specific reactions or other cross reactions cannot be fully studied, false positive results may occur in this test.
7. This test has a low probability of false positive results. Therefore, all positive results must be verified by other method.
8. If the obtained result is questionable, immediately re-test or use other method to test the sample.
9. Test result of this reagent can only be used as an auxiliary means for doctor or other diagnosis. Test result should be combined with other clinical and laboratory data. If test result is not consistent with clinical evaluation, further examination will be required.

LITERATURE REFERENCES

- [1] Wang X, Ling L, Li S, etc. The Diagnostic Value of Gastrin-17 Detection in Atrophic Gastritis: A Meta-Analysis. *Medicine (Baltimore)*. 2016;95(18): e3599.
- [2] Nasrollahzadeh D, Aghcheli K, Sotoudeh M, et al. Accuracy and cut-off values of pepsinogens I, II and gastrin 17 for diagnosis of gastric fundic atrophy: influence of gastritis. *PLoS One*. 2011;6(10): e26957.
- [3] Loong TH, Soon NC, Nik Mahmud NRK, et al. Serum pepsinogen and gastrin-17 as potential biomarkers for pre-malignant lesions in the gastric corpus. *Biomed Rep*. 2017;7(5):460-468. doi:10.3892/br.2017.985.

SYMBOLS


Symbol	interpretation	Symbol	interpretation	Symbol	interpretation
	Consult instructions for use		Tests per kit		Manufacturer
	In Vitro Diagnostic Medical Device		Use-by date		Do not re-use
	Store at 2°C~30°C		Catalogue number		Batch code
	Authorized Representative in the European Community		Caution		



Xiamen Wiz Biotech Co., Ltd.

Address: 3-4 Floor, NO. 16 Building, Bio-medical Workshop, 2030 Wengjiao Xi Road, Haicang District, Xiamen City, Fujian Province, 361026, P.R. China

Tel: +86-592-6808278 Fax: +86-592-6808279

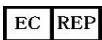

QbD RepS BV

Groenenborgerlaan 16, 2610 Wilrijk, Belgium

CE-01.01-2022.04.13 EN

F1F0185

V1.00 251030

